

1 inch



- Full Face Delivery
- Direct Nasal Delivery
- Nasal Delivery
- Oral Delivery




Infinity⁴⁸¹
DIRECT NASAL MASK



FlexiFit⁴⁰⁷
PREMIUM NASAL MASK



FlexiFit⁴⁰⁶
PETITE NASAL MASK

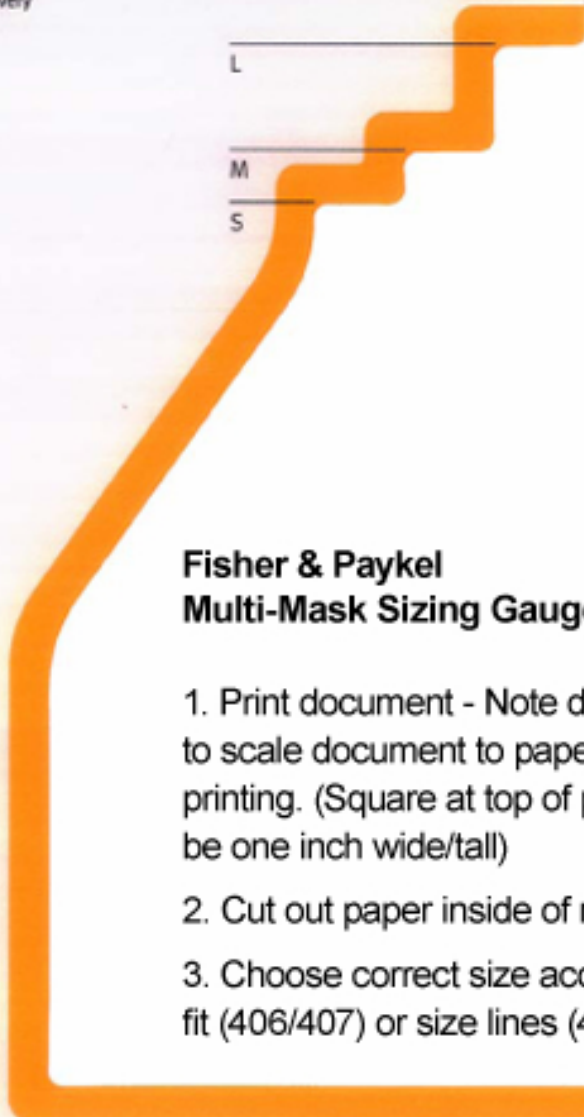

Fisher & Paykel Multi-Mask Sizing Gauge

1. Print document - Note do not choose to scale document to paper size when printing. (Square at top of page should be one inch wide/tall)
2. Cut out paper inside of mask outlines
3. Choose correct size according to fit (406/407) or size lines (431/432/481).

FlexiFit^{431/432}
FULL FACE MASK

185044349REV A

MyCPAP

Presented With Compliments From
www.MyCPAP.com

Fisher & Paykel
HEALTHCARE

www.fphcare.com